

# NEAUVIA

## ZAFFIRO

AWAKE PHYSIOLOGICAL  
BIOSTIMULATION FROM WITHIN



FDA<sup>\*</sup>  
REGISTERED

# ZAFFIRO

## Technology


Zaffiro is a unique combination of non-invasive Infrared Thermolifting and Smart Hydro Technology to brighten and tighten the skin.

### How does it work?

Zaffiro emits infrared light to precisely heat the deep layers of the skin. This thermal effect, known as thermolifting, heats the skin to a controlled temperature up to 65°C, causing collagen and elastin fibers to contract and stimulating fibroblasts to produce new physiological collagen.

Improved collagen structure may help reduce the appearance of fine lines and wrinkles, leading to skin rejuvenation.

Zaffiro's Smart Hydro technology provides gentle cleansing of the skin what helps in improvement of the absorption of active ingredients and enhancing the overall effectiveness of this non-invasive treatment.

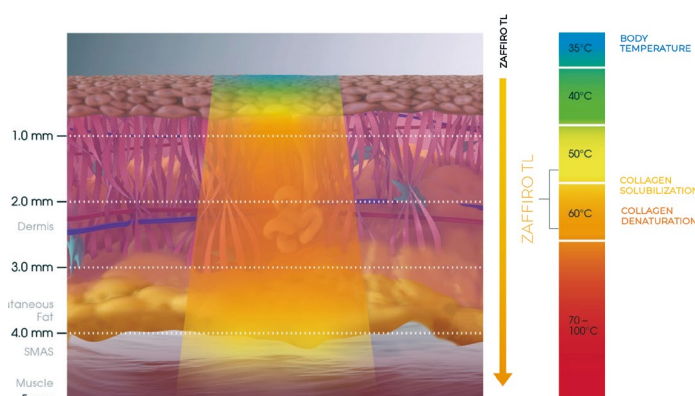

Conceptual schematic representation of infrared technology action.

## Key Benefits

- **Regenerative technology - physiological biostimulation**
- **Non-invasive** treatment for skin tightening
- **Smart Infrared boosting** collagenesis<sup>1</sup>
- **Up to 40% increase in fibroblasts<sup>1</sup> and +30% in skin elasticity<sup>2</sup>, -53% in wrinkles depth and +40% in dermal thickness<sup>3</sup>**
- **Painless patient experience** by continuous cooling system
- **Safe for all skin types** during all year round
- **No downtime**
- **Short treatment time**
- **Delegable device**

<sup>1</sup> Data on file, results 21 days after 1 Zaffiro treatment.

<sup>2</sup> Kubik et al., HA PEGylated Filler in Association with an Infrared Energy Device for the Treatment of Facial Skin Aging: 150 Day Follow-Up Data Report, Pharmaceuticals 2022, 15, 1355.

<sup>3</sup> Data on file, results 90 days after 1 Zaffiro treatment.

## Technical Information

### Infrared Thermolifting:

- ▶ Wavelength Range: Near Infrared (700 to 1800 nm) for a thermo-lifting effect that tightens, lifts, and firms the skin.
- ▶ Sapphire Glass: 6.2 cm<sup>2</sup>
- ▶ Energy density: 5-75 J/cm<sup>2</sup>

### Benefits:

- ▶ Improves firmness, elasticity and texture
- ▶ Decrease in visibility of fine lines and wrinkles

### Smart Hydro Technology

- ▶ Utilizes a high-speed jet of air and 5 watery solutions (soothing skin formula, HA skin health, skin brightening, hair nourishment, acne formula) with microdroplets to gently clean, exfoliate, moisturize, and improve skin texture.

## Handpiece

Smart Infrared handpiece with sapphire glass ensures homogeneous energy delivery to stimulate dermis layer, providing visible results, while continuous cooling system protects the superficial layers of the skin, minimizing burn risk. Smart Hydro handpiece has three streams of accelerated microscopic droplets provide effective and targeted hydroexfoliation.



## Indications

- Loss of firmness
- Wrinkles and fine lines
- Skin texture
- Skin laxity reduction
- Reduction of photoaging of the skin
- Acne treatment (lesions, scars)

\*Regulations and usage conditions for products vary from market to market and the content on this leaflet is not intended for US or Canadian audiences. This leaflet is designed for informational purpose only. For more information do not hesitate to contact your local provider.



The results depicted in these pictures are obtained by certain settings of the device which were tailor-made for the patient of concern. Note that parameters may/must be adjusted per individual and patients may react differently to the treatment based on (but not limited to) medical history, genetics, age, sex, personal motivation, lifestyle, and other factors, and are in no way guaranteed. The product(s) mentioned should not be used without clearance from a physician or health care provider.