

NEAUVIA



Dr. Marta Nowak

Aesthetic medicine,
Poland

2 fillers 5 step technique

REPORT SUMMARY



PATIENT

A 30-year-old female patient, without contraindications,


TREATMENTS

Underwent lip augmentation using the nowak lips 5-step technique. The goal was to improve hydration, smoothness, color, and subtle volume. Treatment included 1 ml **Neauvia Intense Rheology** (22 mg/ml PEG-HA) and 0.2 ml **Neauvia Intense Lips**. Injections were performed

with vertical line and fanning techniques to enhance height, volume, and definition while preserving natural proportions.

RESULTS

The result was refreshed, smoother, and more defined lips with a natural, harmonious appearance.

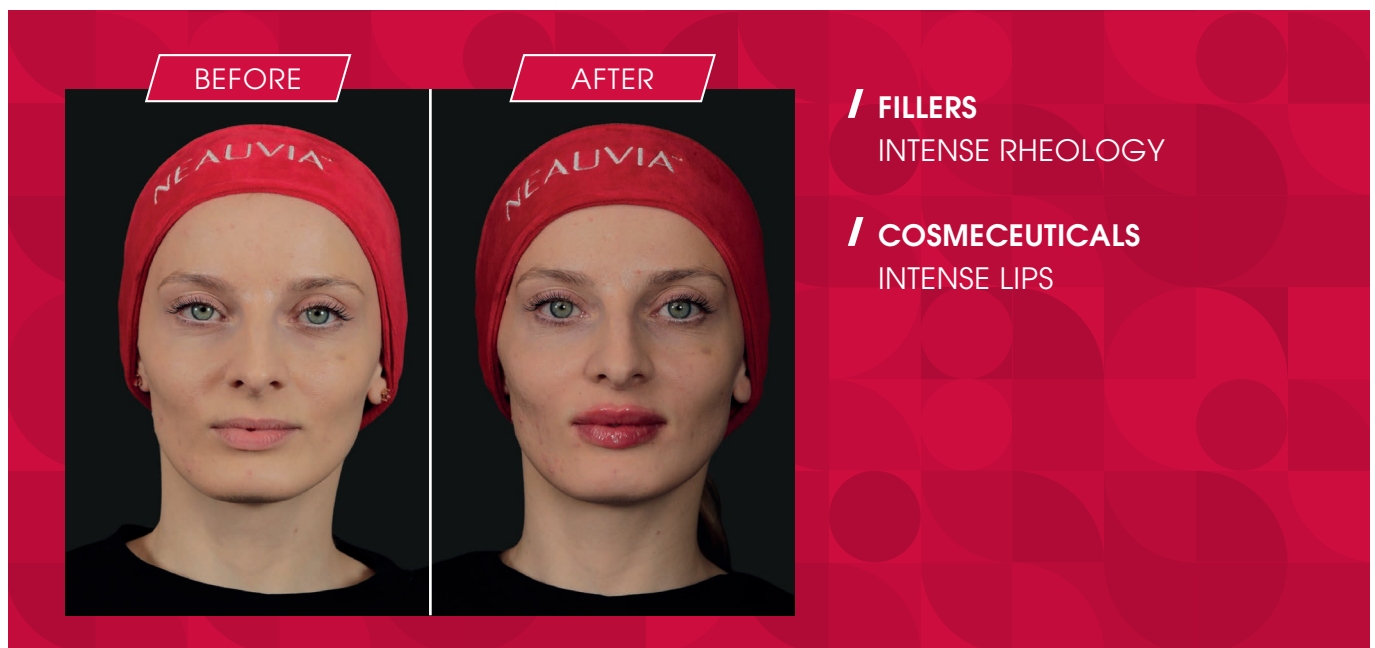

Neauvia products are for trained experts only. Use must follow approved indications and the Instructions for Use (IFU) in accordance to applicable laws and regulations in each country. Physicians are solely responsible for any treatments outside these conditions. The results shown are specific to an individual treatment plan. Treatment decisions must be adapted to each patient, who may respond differently based on numerous factors. Outcomes are variable and not guaranteed.

NEAUVIA *days*

ROME2025