

NEAUVIA



Dr. Agnieszka Walczak-Cieślakowska
Aesthetic medicine,
Poland

Aesthetic Masking of Malocclusion with Cross-Linked Hyaluronic Acid Fillers

REPORT SUMMARY

PATIENT

A 26-year-old female with Class II malocclusion and facial asymmetry sought subtle, non-surgical improvement.

TREATMENTS

A single session of Neauvia PEG-HA fillers was performed: **Neauvia Intense** (2 mL: chin boluses 0.6 mL x3 supraperiosteally, 0.2 mL/side behind mandibular angle, 0.3 mL and 0.2 mL linear along jawline with

cannula) and **Neauvia Intense LV** (1 mL: 0.5 mL/side fanning in marionette lines/ labiomental crease).

RESULTS

Total 3 mL restored contour, improved symmetry, and achieved harmonious, age-appropriate facial enhancement.

BEFORE



AFTER



/ **FILLERS**
INTENSE
INTENSE LV

Neauvia products are for trained experts only. Use must follow approved indications and the Instructions for Use (IFU) in accordance to applicable laws and regulations in each country. Physicians are solely responsible for any treatments outside these conditions. The results shown are specific to an individual treatment plan. Treatment decisions must be adapted to each patient, who may respond differently based on numerous factors. Outcomes are variable and not guaranteed.

NEAUVIA *days*
ROME2025