

# NEAUVIA



## Be you new skin 6.0 protocol

**Dr. Viviane Rabelo**

DDS, MSc  
Brazil

### REPORT SUMMARY



#### PATIENT


A 67-year-old woman, phototype VI, presented with sagging, dull skin and deep creases.

#### TREATMENTS

She was treated with the Be You New Skin 6.0 Protocol using the **LaserMe** 1470 nm fractional non-ablative diode laser, safe for higher phototypes as it targets water instead of melanin. Four sessions (16 mJ/MTZ, 2.5 mm spacing, 70 shots each) were performed every 15 days.

#### RESULTS

Results showed marked wrinkle reduction, firmer contours, improved texture and skin tone. 3D analysis confirmed dermal remodeling, and the patient reported high satisfaction.

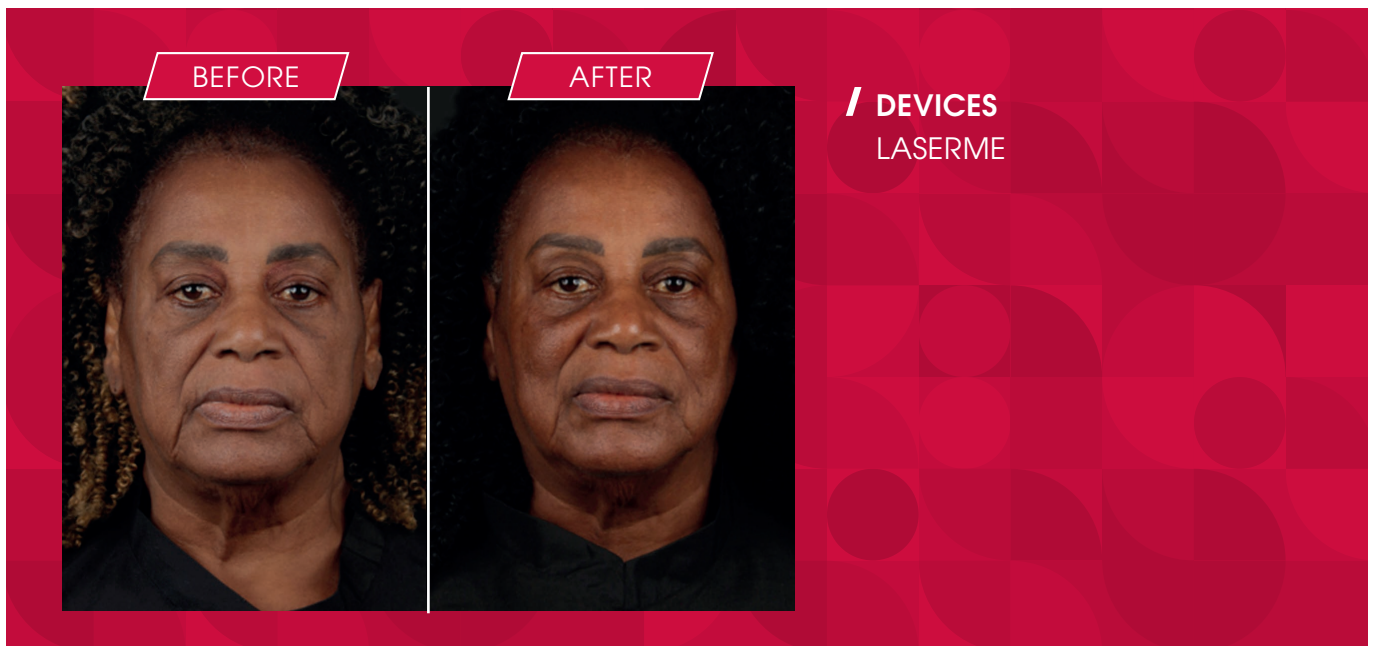

Neauvia products are for trained experts only. Use must follow approved indications and the Instructions for Use (IFU) in accordance to applicable laws and regulations in each country. Physicians are solely responsible for any treatments outside these conditions. The results shown are specific to an individual treatment plan. Treatment decisions must be adapted to each patient, who may respond differently based on numerous factors. Outcomes are variable and not guaranteed.