

NEAUVIA



**Dr. İlker Yörük
& Dr. Meryem Özlem Öztürk**
Medical Aesthetic & Dermatology,
Turkey

REPORT SUMMARY



PATIENT

A 47-year-old female with progressive midface volume loss due to weight fluctuations and no prior filler history sought lifting without excess volume.

TREATMENTS

The protocol began with **Neauvia Hydro Deluxe** (2.5 ml point by point) for skin hydration and preparation. In the second session, **Neauvia Intense** (1 ml, bolus and fan) was applied to the maxilla,

while **Neauvia Stimulate** (2 ml, bolus and fan) targeted the zygomatic arch and mandibular corner. **Zaffiro** infrared tightening (30 mJ, 90 passes) further enhanced elasticity.

RESULTS

Results showed natural lifting, skin firmness, and refreshed contours with minimal downtime.

BEFORE



AFTER



/ FILLERS
HYDRO DELUXE
STIMULATE
INTENSE

/ DEVICES
ZAFFIRO

Neauvia products are for trained experts only. Use must follow approved indications and the Instructions for Use (IFU) in accordance to applicable laws and regulations in each country. Physicians are solely responsible for any treatments outside these conditions. The results shown are specific to an individual treatment plan. Treatment decisions must be adapted to each patient, who may respond differently based on numerous factors. Outcomes are variable and not guaranteed.

NEAUVIA *days*
ROME2025