

# NEAUVIA



**Dr Izumrud Ramazanova  
Kurbankadieva**

Aesthetic Medicine,  
Spain

## Bio-regeneration for lasting impact



### REPORT SUMMARY

#### PATIENT

Healthy 57-year-old woman (Fitzpatrick IV) with hypertrophic tissue and moderate skin laxity, plus texture alterations linked to chronoaging and photoaging.

#### TREATMENTS

Session 1: Smart Hydro (WaterPeeling), ColdPad, **Neauvia Hydro Deluxe**, non-crosslinked HA with low-concentration CaHA, injected as small papulae using a 32G 2 mm needle (2.5 ml full-face).  
Session 2 (1 week later): Smart Hydro >

**Zaffiro** infrared (40 J, 40 pulses, 30 min) > Smart Hydro > **LaserMe** (35 mJ / 1.5 mm spacing) > **Instant Recovery Mask** > **Ceramide Shield Cream**.

#### RESULTS

The treatment delivered visible improvements in texture, pore size, and wrinkles, with excellent tolerance. This case exemplifies bioregeneration at its most refined: a layered, science-led protocol where preparation meets precision - and results continue to grow long after treatment.

BEFORE



AFTER



**/ FILLERS**  
HYDRO DELUXE

**/ DEVICES**  
ZAFFIRO  
LASERME

**/ COSMECEUTICALS**  
INSTANT RECOVERY MASK  
CERAMIDE SHIELD CREAM

Neauvia products are for trained experts only. Use must follow approved indications and the Instructions for Use (IFU) in accordance to applicable laws and regulations in each country. Physicians are solely responsible for any treatments outside these conditions. The results shown are specific to an individual treatment plan. Treatment decisions must be adapted to each patient, who may respond differently based on numerous factors. Outcomes are variable and not guaranteed.