

**Dr. Sabina-Eliza Băloi**Plastic surgeon,  
Romania

## Comprehensive vaginal restoration

### REPORT SUMMARY



#### PATIENT

A 45-year-old woman with atrophic labia majora, loose, wrinkled skin, hyperpigmentation, dryness, and dyspareunia.

#### TREATMENTS


Received **Neauvia Intense Rose** 28 mg/ml (1.1 ml L, 0.9 ml R) via fanning and retrograde small bolus for augmentation, combined with internal and external **Sectum RF** (20 W, 6 min, 38–42 °C, 4

passes; 8 W, 10 min, 38–42 °C, multiple passes) and at-home **Neauvia Rejuvenation Rose Gel**.

#### RESULTS

Treatment improved hydration, tissue tone, elasticity, and sexual well-being, providing safe and lasting intimate rejuvenation.

BEFORE

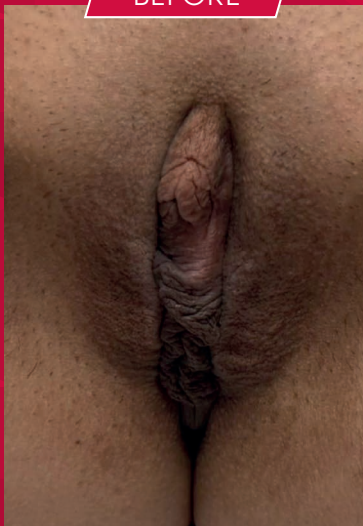

AFTER



/ **FILLERS**  
INTENSE ROSE

/ **DEVICES**  
SECTUM RF

/ **COSMECEUTICALS**  
REJUVENATION ROSE GEL

Neauvia products are for trained experts only. Use must follow approved indications and the Instructions for Use (IFU) in accordance to applicable laws and regulations in each country. Physicians are solely responsible for any treatments outside these conditions. The results shown are specific to an individual treatment plan. Treatment decisions must be adapted to each patient, who may respond differently based on numerous factors. Outcomes are variable and not guaranteed.

**NEAUVIA** *days*  
ROME2025