

NEAUVIA



Dr. Patrycja Kieblesz
Aesthetic medicine Doctor, Dentist,
Poland

Clinical case in the use of Neauvia hyaluronic acid

REPORT SUMMARY



PATIENT

A healthy 45-year-old woman sought mid- and lower-face restoration and lip contour enhancement.

TREATMENTS

Treatment included **Neauvia Intense 2** ml, **Neauvia Stimulate 1** ml, and **Neauvia Intense Lips 1** ml. Cheek volumetry used 22G cannula (0.7 ml per side), jawline and angle contouring with 22G cannula

(0.7 ml per side), and lips were shaped via fan technique with 31G needle (0.6 ml). Local anesthesia was applied.

RESULTS

Immediate results showed improved facial contour, defined cheekbones, and enhanced lips with minimal product, achieving a natural and satisfying outcome.

BEFORE



AFTER



/ FILLERS
STIMULATE
INTENSE
INTENSE LIPS

Neauvia products are for trained experts only. Use must follow approved indications and the Instructions for Use (IFU) in accordance to applicable laws and regulations in each country. Physicians are solely responsible for any treatments outside these conditions. The results shown are specific to an individual treatment plan. Treatment decisions must be adapted to each patient, who may respond differently based on numerous factors. Outcomes are variable and not guaranteed.

NEAUVIA *days*
ROME2025