

NEAUVIA



Dr. Ivan Medrano
Dermatologist,
Peru

Hyaluronic acid for masculinizing oval-shaped faces

REPORT SUMMARY



PATIENT

A 40-year-old male (skin type IV) presented with dissatisfaction regarding facial definition and sagging.

TREATMENTS

A single session of **Neauvia Intense Man** (PEG-HA 28 mg/ml) was performed using supraperiosteal and subcutaneous injections. Product distribution: zygomatic

arch 0.5 ml/side, malar 0.5 ml/side, chin 1 ml, gonion 0.5 ml/side, jawline 0.5 ml/side, mandibular ramus 0.5 ml/side.

RESULTS

The treatment achieved enhanced contouring of the mid and lower face with natural, masculine definition and high patient satisfaction.

BEFORE



AFTER



/ **FILLERS**
INTENSE MAN

Neauvia products are for trained experts only. Use must follow approved indications and the Instructions for Use (IFU) in accordance to applicable laws and regulations in each country. Physicians are solely responsible for any treatments outside these conditions. The results shown are specific to an individual treatment plan. Treatment decisions must be adapted to each patient, who may respond differently based on numerous factors. Outcomes are variable and not guaranteed.

NEAUVIA *days*
ROME2025