

# NEAUVIA



**Dr. Gjorgi Stavrikj**  
Plastic surgery  
North Macedonia

## Lip, chin, jawline volume restoration and defining

### REPORT SUMMARY



#### PATIENT

A 47-year-old woman (Fitzpatrick III) with moderate volume loss in the lower face, weak chin and jawline, and asymmetrical lips.

#### TREATMENTS


Underwent a single-session treatment using **Neauvia Intense** (28 mg/ml) for chin and mandibular enhancement and **Neauvia LV** (26 mg/ml) for lip volumizing. Lips were injected with linear retrograde

technique (0.2–0.3 ml per quadrant), gonial angles and chin with bolus (0.2–0.3 ml), and mandibular body with cannula (0.5 ml per side).

#### RESULTS

The procedure restored symmetry, volume, and contour, improved lower third proportions, and provided hydration, achieving natural results without complications.

BEFORE

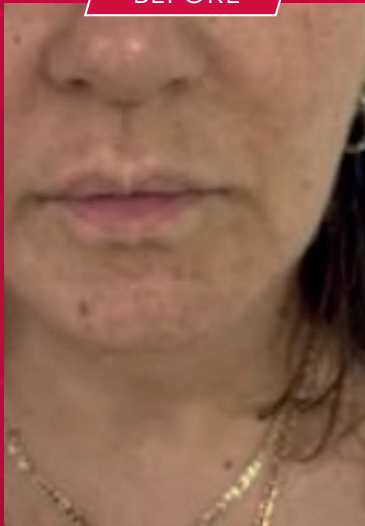

AFTER



**/ FILLERS**  
INTENSE  
INTENSE LV

Neauvia products are for trained experts only. Use must follow approved indications and the Instructions for Use (IFU) in accordance to applicable laws and regulations in each country. Physicians are solely responsible for any treatments outside these conditions. The results shown are specific to an individual treatment plan. Treatment decisions must be adapted to each patient, who may respond differently based on numerous factors. Outcomes are variable and not guaranteed.

**NEAUVIA** *days*  
ROME2025