

# NEAUVIA



**Dr. Armand Delgado**  
Aesthetic Doctor,  
Spain

## Nlines + LaserMe + Intense Rheology (fern pattern) + Stimulate

### REPORT SUMMARY



#### PATIENT

A 46-year-old female with Fitzpatrick II skin and midface volume loss.

#### TREATMENTS

Underwent the **Nlines protocol** combining **Neauvia Stimulate** (2 mL per session, fanning, subdermis, 2 sessions), **Neauvia Intense Rheology** (0.5 mL per session, fern pattern, deep dermis, 3 sessions), and **LaserMe** 1470 nm fractional

laser (35 mJ/point, 1–2 passes, 1.5 mm spacing, 3 sessions).

#### RESULTS

Treatments spaced 4 weeks apart restored volume, improved structural support, enhanced skin firmness, reduced wrinkles, refined texture, and delivered natural, long-lasting rejuvenation.

BEFORE



AFTER



**/ FILLERS**  
STIMULATE  
INTENSE RHEOLOGY

**/ DEVICES**  
LASERME

Neauvia products are for trained experts only. Use must follow approved indications and the Instructions for Use (IFU) in accordance to applicable laws and regulations in each country. Physicians are solely responsible for any treatments outside these conditions. The results shown are specific to an individual treatment plan. Treatment decisions must be adapted to each patient, who may respond differently based on numerous factors. Outcomes are variable and not guaranteed.