

# NEAUVIA



**Dr. Cyril Mitchel Agan**  
Aesthetic medicine,  
Philippines

## Sexy jaw: male jawline contouring using combination PEG-HA Filler

### REPORT SUMMARY



#### PATIENT

A 27-year-old male with a retruded chin and weak lower face, healthy after surgical correction of congenital ASD, underwent non-surgical jawline augmentation.

#### TREATMENTS

**Neauvia Intense** (PEG-HA 28 mg/ml) was used for deep supraperiosteal projection of the chin (0.8 ml) and gonial angle (0.2 ml), combined with **Neauvia Intense LV**

(PEG-HA 28 mg/ml) for subcutaneous jawline and chin contouring.

#### RESULTS

Using needle boluses and retrograde cannula threads, the "Sexy Jaw" technique achieved natural masculine definition and a balanced lower face with high patient satisfaction.

BEFORE



AFTER



**/ FILLERS**  
INTENSE  
INTENSE LV

Neauvia products are for trained experts only. Use must follow approved indications and the Instructions for Use (IFU) in accordance to applicable laws and regulations in each country. Physicians are solely responsible for any treatments outside these conditions. The results shown are specific to an individual treatment plan. Treatment decisions must be adapted to each patient, who may respond differently based on numerous factors. Outcomes are variable and not guaranteed.