

NEAUVIA



Dr. Eva Guisantes
Plastic surgeon,
Spain

Treatment of facial asymmetry and masculinization with Neauvia fillers

REPORT SUMMARY



PATIENT

A 26-year-old male with facial asymmetry and hypoplasia was treated with PEG-HA fillers to enhance chin, malar projection, and jawline definition.

TREATMENTS

Treatment included **Neauvia Intense LV** 1 cc for the midface, **Neauvia Intense Man** 2 cc for the chin, malar, and mandibular angle, and **Neauvia Stimulate Man** 2 cc for the lateral

cheek. Fanning and retrograde cannula injections with supraperiosteal boluses achieved targeted augmentation.

RESULTS

The 5 cc total filler corrected asymmetry, improved projection, and defined the jawline and chin safely, with high patient satisfaction.

BEFORE



AFTER



/ FILLERS
STIMULATE MAN
INTENSE MAN
INTENSE LV

Neauvia products are for trained experts only. Use must follow approved indications and the Instructions for Use (IFU) in accordance to applicable laws and regulations in each country. Physicians are solely responsible for any treatments outside these conditions. The results shown are specific to an individual treatment plan. Treatment decisions must be adapted to each patient, who may respond differently based on numerous factors. Outcomes are variable and not guaranteed.